

**Advanced Medical Innovations™**

**Mission Industrial Park**

8741 Shirley Avenue, Northridge, CA 91324 USA Tel: (818) 701-7180 Fax: (818) 701-9708 Fax to Email: (818) 350-5738

Internet Address: <http://www.amiwelisten.com> E-Mail Address: [sales@amiwelisten.com](mailto:sales@amiwelisten.com)

**SECURE-IT-Y™**

**Instrument Counter and Organizer with Attached Wire - Patent Pending**

P/N 14301/ 14321

**Reusable / Autoclavable**

**INSTRUCTION FOR USE**

1. Lay instruments in the slots from 1-15 (Tips of instruments facing away) for Counter or next to each other in the Organizer (about 30 instruments). Move the attached wire forward and over the instruments ring to lock in place (wire will be held in place by friction fit).
2. Make sure all instruments are in the open position (ratchet loose) or as per your facility's policy.
3. For those who use the L-shaped stringer, insert the long part of the wire through the top finger ring of the instruments. Lift the L-shaped wire to insure all instruments are in the open position and insert the flattened part of the wire (facing down ) into the slotted hole in the plastic instrument rack until secure.
4. Repeat steps 4 and 5 if using a second stringer from the opposite side (optional).
5. The instrument Counters / Organizers can now be sterilized with the instruments according to your facilities procedures.

TO HANDLE INSTRUMENTS FOLLOWING STERILIZATION

1. Remove L-shaped stringer (s), if used, by pulling it up from the slotted hole and sliding it away from the instruments. Snap the L-shaped stringer(s) alongside the front snap hooks on the surface of the instrument rack for safe keeping while not in use.
2. Move the attached wire back into the open position for handling the instruments in and out of the rack during use.
3. Utilize the instrument rack as a counter by filling the 15 slots with instruments and counting them in totality to compare with the original count on the instrument sheet. Only when missing an instrument, count by type or follow your facility's procedures for instrument counting.

Note: The Instrument Counter and Organizer and stainless steel wires and stringers are compatible with instrument cleaning, processing and sterilization equipment and materials, including, ultrasonic cleaners, washer tunneler, steam sterilizers, etc. Use your facility's instrument cleaning procedures with these products.

Prepared by: Shardel Doherty

Approved by QA: B

Date: 12/15/2011

Approved by Eng: MA

Date: 12/15/2011

Form #IFU0004 Rev: A